


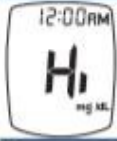



	Test Strip Error, Very high blood glucose result - higher than 600mg/dL	Retest with new Test Strip. If error persists, call for assistance. If you have symptoms such as fatigue, excess urination, thirst, or blurry vision follow your healthcare professional's advice for high blood glucose.
	Test Strip Removed During Test	Retest with new test strip. Make sure result is displayed before removing test strip.
	Communication Error	Call for assistance.
	Low or Dead Battery	Low: About 50 tests can be done before battery dies. Dead: Battery Symbol appears and beeps before meter turns off.
	WARNING!! Out of Range - High Results > 600 mg/dL	<div style="display: flex; justify-content: space-between; align-items: center;"> <div style="text-align: center;">  </div> <div style="text-align: center;"> <p>WARNING!! Retest with new test strip. If result is still "Hi" (High) or "Lo" (Low) contact Doctor immediately.</p> </div> <div style="text-align: center;">  </div> </div>
	Out of Range - Low Results < 20 mg/dL	